


Smart Fitness Bracelet

User's Manual

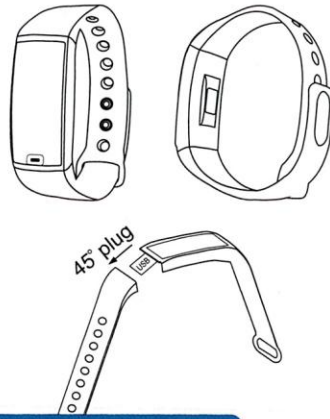

User's Manual



Support device version



Android 4.4 +



IOS 8.2 +



Bluetooth 4.0 +

Quick Start Guide

1. Download WearHealth
The wristband requires application support before it can be connected to the cell phone. Before downloading the application, please check the requirements of the mobile phone system hardware platform.

Searching method 1: Scan the QR code to download



IOS App



Google play



Android App

2. Product activation and charging

The initial use of the product requires a charge to activate the product. And please make sure the battery is normal by charging. The wristband will automatically shut down. (The charging method is shown below: Plug in the regular DC interface directly with USB. Requirements: 5V ~ 500mA)



USB Charging port

Please read this user's manual carefully before use

Thank you for purchasing this product. We highly appreciate any feedback which helps us to improve our products. Please keep this instruction manual for future reference.

Start to use

Key function

Click: function switch

Long press: activate, mode switch



Once the phone is successfully bound, start recording and analyzing your movement, sleep, and display mode such as touch, click and switch.

Operating Instructions

- Please fully charge the bracelet before using, and it will launch automatically after charge.
- Power ON/OFF, Switch to the power off page shown as below image:



model version ID



Click on the select



Long press to turn it off

Click "YES" and long press the power button to turn OFF.

Short press the power button it will turn ON. During the charging, it will auto launch.

Bracelet Functions

1. Normal mode display



Running



Running message



Calories



Distance



PPG heart rate measurement



Blood Pressure



Message



Steps

2. Clock interface and search cellphone display



Standard interface



Long press to search cellphone

3. Continuous heart rate interface display



PPG heart rate measurement



Heart rate value (Vibration prompt measurement is over)

Warranty

The product is guaranteed by the manufacture for a period of 1 year from the date of purchase the guarantee does not cover defects arising from accident damage misuse or wear and tear and is available only to the original purchaser of the product.

Specifications

- Dimensions: 230*11*20mm
- Product weight: 20g
- Display screen: 0.96" IPS
- Resolution: 128*64dpi
- CPU: nRF 52832
- Battery: 85mAh
- Standby time: 7-12 days
- Working time: 3-5 days
- Waterproof: IP67
- Body material: ABS+PC
- Band material: TPU
- Compatibility: IOS8.2+Android 4.4

Contents of the page

- 1x Smart Bracelet
- 1x User's Manual

FAQ

1. The wristband cannot be found when pairing?

Please make sure your phone is Bluetooth enabled. And when the phone is pairing in the "support range", please make sure that the distance between the wristband and the cellphone is within 0.5 meters. The Bluetooth communication range is within 10 meters after pairing.

2. Why is Bluetooth not connected occasionally?

Some mobile phone's Bluetooth service will have an exception when it is restarted, which leads to such a situation. Usually you can reset the Bluetooth or restart the phone to establish a normal connection.

3. How to restore the factory default settings?

After the wristband is connected to your cellphone, please enter the "My wristband management" in the application and choose "restore factory settings".

4. How to update the firmware of the wristband?

After the wristband is connected to your cellphone, please enter the "wristband management" in the application and choose "firmware update".